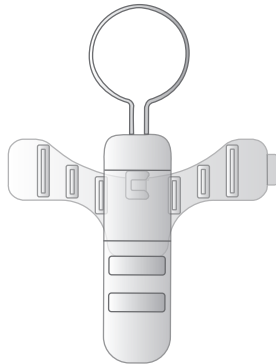
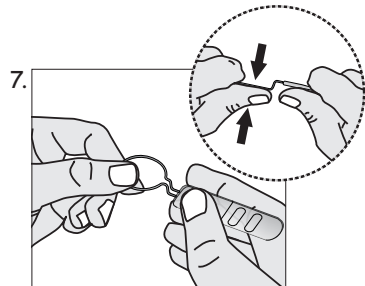
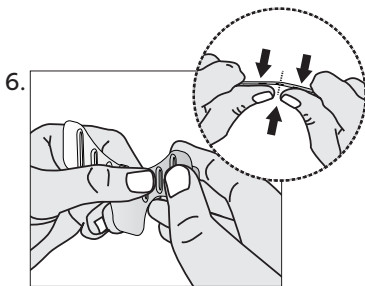
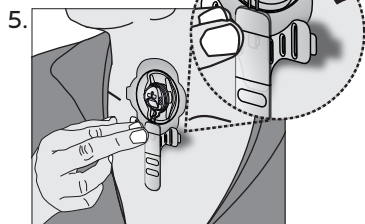
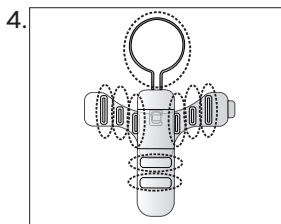
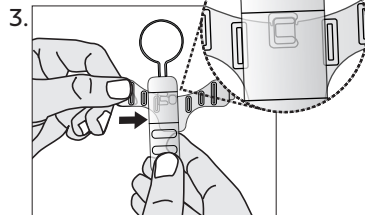
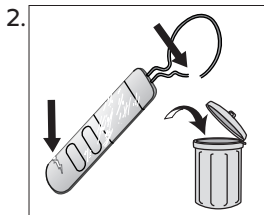
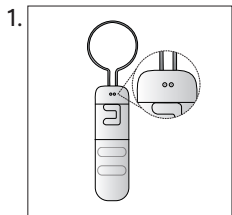
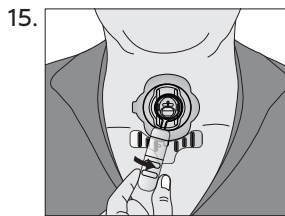
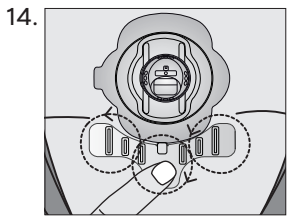
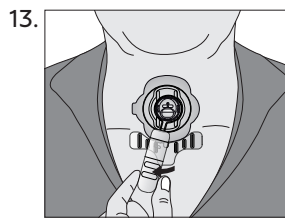
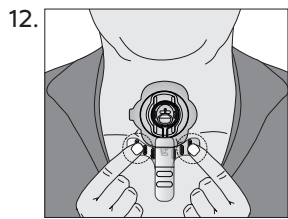
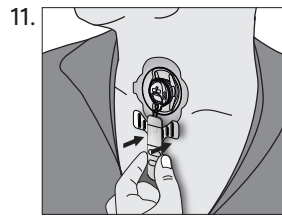
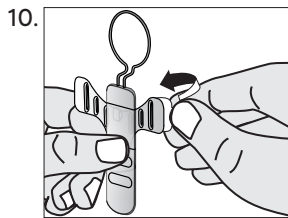
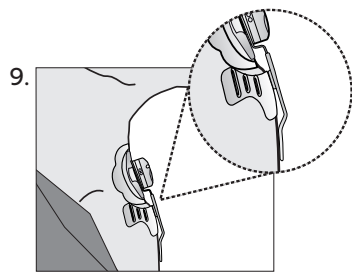
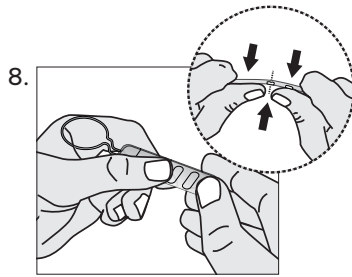
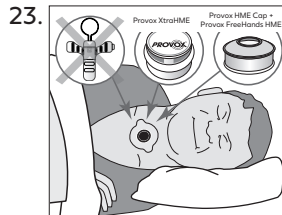
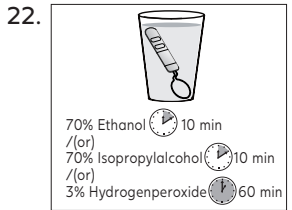
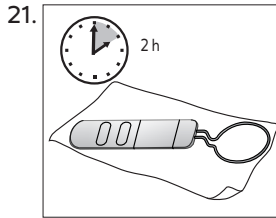
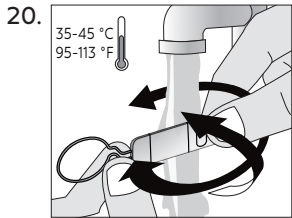
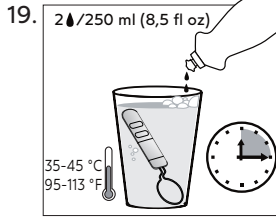
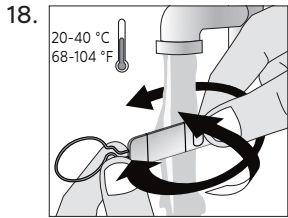
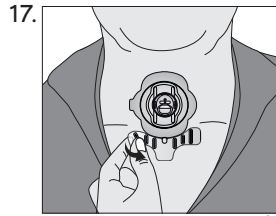
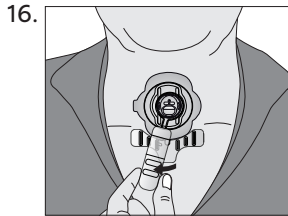
PROVOX[®] FreeHands Support[™]

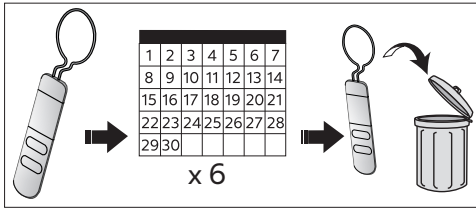
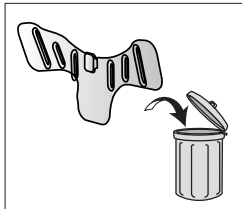
24.

25.

Disclaimer

Atos Medical offers no warranty – neither expressed nor implied – to the purchaser hereunder as to the lifetime of the product delivered, which may vary with individual use and biological conditions. Furthermore, Atos Medical offers no warranty of merchantability or fitness of the product for any particular purpose.

Contents

EN - ENGLISH.....	13	PL - POLSKI.....	73
DE - DEUTSCH.....	16	RO - ROMÂNĂ.....	77
NL - NEDERLANDS.....	20	HR - HRVATSKI.....	81
FR - FRANÇAIS.....	24	SR - SRPSKI.....	84
IT - ITALIANO.....	28	EL - ΕΛΛΗΝΙΚΑ.....	87
ES - ESPAÑOL.....	32	TR - TÜRKÇE.....	91
PT - PORTUGUÊS.....	35	HY - ՀԱՅՅԵՐԵՄ.....	94
SV - SVENSKA.....	39	RU - РУССКИЙ.....	98
DA - DANSK.....	42	MS - BAHASA MELAYU.....	102
NO - NORSK.....	45	JA - 日本語.....	106
FI - SUOMI.....	48	KO - 한국어.....	109
IS - ÍSLENSKA.....	51	ZHTW - 繁體中文.....	112
ET - EESTI.....	54	HE - עברית.....	117
LT - LIETUVIŲ KALBA.....	57	AR - العربية.....	120
CS - ČESKY.....	60	ORDERING INFORMATION.....	121
HU - MAGYAR.....	64		
SK - SLOVENČINA.....	67		
SL - SLOVENŠČINA.....	70		

1. Descriptive Information

1.1 Indications for use

Provox FreeHands Support provides support to the Provox Adhesive when using a Provox hands-free speaking valve after total laryngectomy. The device is a single patient use device with a single use adhesive.

1.2 CONTRAINDICATIONS

- Do not use Provox FreeHands Support on breached skin.
- Provox FreeHands Support is NOT intended to be in place during MRI examination (Magnetic Resonance Imaging), or during the period of Radiation Therapy treatment.

1.3 Description of the device

The Provox FreeHands Support system consists of two parts; Provox FreeHands Support and Provox FreeHands Support Adhesive.

The reusable Provox FreeHands Support consists of a plastic part and a supporting metal ring. Provox FreeHands Support is available in three different variants; 'Flat', 'Medium' and 'Deep'. On the reverse of the plastic part, just under the ring, there are 1, 2 or 3 dots visible (fig. 1). The number of dots represents the depth of the ring.

1 dot = Provox FreeHands Support Flat (REF 8021)

2 dots = Provox FreeHands Support Medium (REF 8022)

3 dots = Provox FreeHands Support Deep (REF 8023)

The three variants, as well as the ability to adjust the metal ring and plastic parts, enable a personal fit. Note, the metal ring contains nickel, however the amount of free nickel in the material is below the level which would normally give a reaction in people with a nickel-allergy.

Provox FreeHands Support Adhesive consists of an adhesive tape with a peel-off liner and a plastic plate. Provox FreeHands Support Adhesive is a single-use adhesive that is attached to the skin, beneath the Provox Adhesive.

Caution: Use only original components. Other accessories may cause product malfunction.

1.4 WARNINGS

- The adhesive may irritate the skin. If irritation occurs, stop using the adhesive and consult your clinician.
- Reuse of the Provox FreeHands Support Adhesive, by either yourself or someone else, or reuse of the Provox FreeHands Support by another person may cause transfer of micro-organisms which can lead to infections.

2. Instructions for Use

2.1 Preparation

Make sure the skin is clean and dry. To improve adhesion you may apply skin protection products, such as Provox Skin Barrier, or adhesive supporting products such as Provox Silicone Glue, in accordance with their instructions for use.

2.2 Inspection procedures

Always inspect Provox FreeHands Support to make sure that it is not broken or worn out, see fig. 2. If any faults are found during this check, the device must be discarded and not used.

2.3 Operating instructions

Preparation

Assemble the device by sliding FreeHands Support to the right over FreeHands Support Adhesive to fasten the locking mechanism (fig. 3).

Several areas of the device are bendable for personal fit (fig. 4). To adjust FreeHands Support to your anatomy, place the ring of FreeHands Support over the Provox hands-free speaking valve and hold the device to your sternum (chest bone). Identify any areas that require adjustment (fig. 5). Remove the device and bend FreeHands Support and FreeHands Support Adhesive at the indicated folding areas to get a good personal fit (fig. 6-9).

Attachment

Keep FreeHands Support attached to FreeHands Support Adhesive. Remove the liner from the adhesive (fig. 10) and place the ring over the Provox hands-free speaking valve (fig. 11). When aligned, press the FreeHands Support Adhesive against your skin. Remove FreeHands Support and rub the adhesive in gentle circles to fix it to the sternum (fig. 12-14). Reattach FreeHands Support by placing the ring over the Provox hands-free speaking valve and slide it to your left into the locking mechanism while keeping the ring still (fig. 15).

Use

FreeHands Support can easily be removed and reattached as needed during the day. It is not necessary to remove FreeHands Support Adhesive each time (fig. 15-16).

Removal

Remove Provox FreeHands Support by sliding FreeHands Support to your right while keeping the ring still over the Provox hands-free speaking valve (fig. 16).

To remove FreeHands Support Adhesive grab at one end and pull it gently from the skin (fig. 17). If required use Provox Adhesive Remover.

2.4 Cleaning and Disinfection

Clean FreeHands Support daily according to fig. 18-21 using drinking water and washing up liquid.

Disinfection might be needed if FreeHands Support looks dirty or there has been a risk of contamination. If disinfection is needed, first clean the device as described above and then disinfect the device according to fig. 22.

Disinfect FreeHands Support (fig. 22) with one of the following methods:

- Ethanol 70% for 10 minutes
- Isopropylalcohol 70% for 10 minutes
- Hydrogenperoxide 3% for 60 minutes

Caution: Do not clean or disinfect by any other method as this might cause product damage and injury.

2.5 Storage instructions

When not in use, clean and disinfect the device as described above and then store it in a clean and dry place at room temperature. Protect from direct sunlight.

2.6 Device lifetime

Provox FreeHands Support is a single patient use device and should be replaced after 6 months of use (fig. 24). Check the device for signs of damage before each use according to the Inspection procedures (section 2.2).

Provox FreeHands Support Adhesive is for single use (fig. 25). It is disposable and shall be discarded after use.

2.7 Disposal

Always follow medical practice and national requirements regarding biohazard when disposing of a used medical device.

3. Additional Information

3.1 Compatibility with MRI Examination

Provox FreeHands Support is MRI unsafe. This device contains metallic parts and should therefore not be used during an MRI examination.

3.2 Travel or International use

Provox FreeHands Support contains metallic parts; please take this into account when traveling.

3.3 Order information

See end of this Instructions for Use.

4. Reporting

Please note that any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the national authority of the country in which the user and/or patient resides.

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© Coloplast A/S. All rights reserved.
Atos and the Atos Medical logo are trademarks of Coloplast A/S.
Atos Medical UK Ltd Tottle Road
Atos Medical AB